

Donning PPE – Droplet & Contact Precautions

<p>1) PREPARE</p> <ul style="list-style-type: none"> Remove watches, jewelry and any personal items. Ensure ID lanyard is covered or removed Secure hair back 		
<p>2) ASSEMBLE SUPPLIES</p> <ul style="list-style-type: none"> Long sleeved gown, surgical/ procedure mask, eye protection, gloves, hand hygiene supplies 		
<p>3) PERFORM HAND HYGIENE</p>		
<p>4) DON GOWN</p>		
<p>Secure neck ties</p> 	<p>Secure waist ties</p> 	<p>Wear gown close to neck</p> 
<p>5A) DON MASK AND EYE PROTECTION</p>		
<p>Place straps over ears / Mold mask to nose /Fit mask under chin</p>		
<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;"> <p>Safety glasses Option</p>  </div> <div style="text-align: center;"> <p>Face Shield Option</p>  </div> </div>		
<div style="display: flex; justify-content: space-around;">    </div>		
<p>5B) DON MASK WITH FACE SHIELD ATTACHED</p>		
<p>Place straps over ears / Mold mask to nose /Fit mask under chin</p>		
<div style="display: flex; justify-content: space-around;">    </div>		
<p>6) DON GLOVES</p> <p>Gloves over cuff</p> 	<p>READY</p> <div style="display: flex; justify-content: space-around;">    </div>	